

如何將植體資料庫匯入 3Shape

How to load the Anker library into 3Shape

1. 將資料庫下載後解壓縮

Load the library from Anker Website, and unzip the file

2. 打開 **3Shape Dental System Control Panel**

Click on 3Shape Dental System Control Panel.



3. 點擊**導入/導出**功能

Click on Import/ Export



工具

订购管理 (加密狗)

导入/导出

下载中心

培训中心

3Shape Communicate

4. 點擊導入材料

Click on Import Material.



5. 導入數據庫資料包並結束

Load the Library and finish.

