

# 如何將植體資料庫匯入 Exocad


How to load the Anker library into exocad

1. 將資料庫下載後解壓縮

Load the library from Anker Website, and unzip the file

2. 在您安裝 Exocad 軟體的地方，找到資料夾 DentalCADApp

Find the file "DentalCADApp" in your exocad installation.

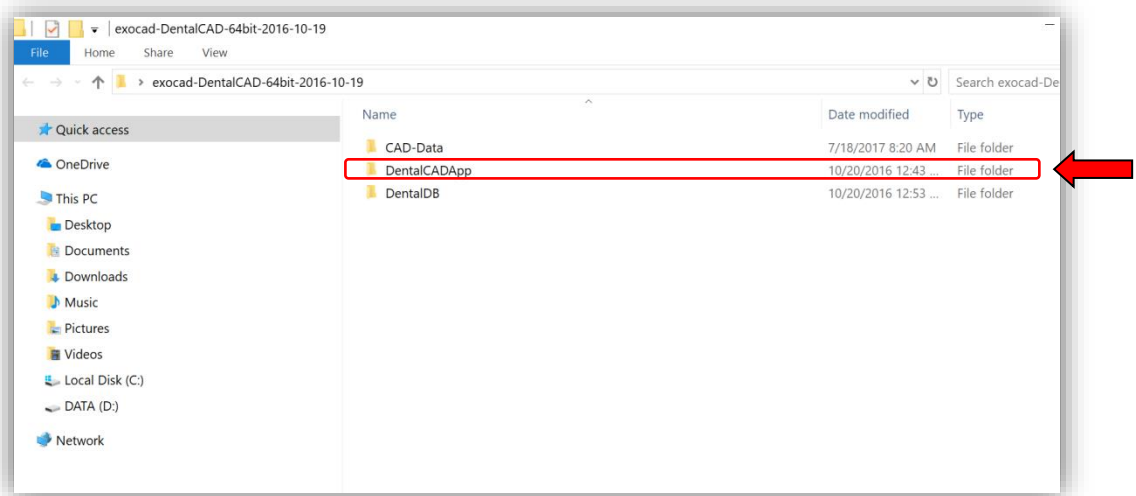

3. 點擊進 DentalCADApp→library→implant，將 Anker 資料夾複製貼上到 implant 資料夾中即完成

Copy the Anker library to DentalCADApp/library/implant



4. 資料庫成功顯示 Anker 植體  
Start using CAD software.

